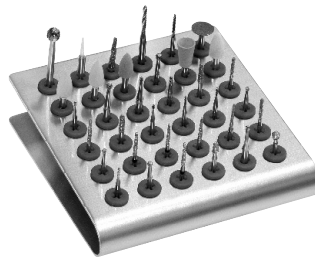
NTI-Kahla GmbH • Rotary Dental Instruments
 Im Camisch 3 • D-07768 Kahla/Germany
 Tel. +49(0)36 424-573-0 • Fax +49(0)36 424-573-29
 e-mail: nti@nti.de • www.nti.de

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GB Instructions for use for thermal disinfection bur block

Contents:

1. Product Description

2. Indication of bur block: Thermal disinfection
3. Contraindication
4. Cleaning
5. Reprocessing

1. Product Description

Dear customer,

The thermal disinfectant bur block may only be used by dentists or trained hygiene personnel. Please carefully read through this product information prior to using the thermal disinfectant bur block.

The thermal disinfectant bur block was specially designed for use with medical devices and must be used in accordance with the recommended instructions for use. The manufacturer accepts no liability for damage that results from not complying with these instructions or due to inappropriate use of the system.

2. Indication of bur block: Thermal disinfection

The bur block has been designed for use in conventional thermal disinfection equipment. The silicone holders are suitable for FG, contra-angle and straight handpiece instruments.

The instruments that are to be cleaned in the thermal disinfectant are inserted into the silicone holder. To prevent cross-contamination, it is recommended to wear protective gloves. Insert the instruments only so that the end of the working part protrudes about 3-4 mm above the silicone holder.

After filling, fix the block with the clamps located on the base to a drawer of the thermal disinfectant.

It is strongly recommended that only cleaning agents and accessory liquids that are prescribed by the manufacturer of the equipment are used. Other agents, particularly agents that are intended for a dishwasher, are not suitable for disinfection and will damage instruments and the bur block.

3. Contraindication

The bur block is not autoclavable.

4. Cleaning:

The thermal disinfectant bur block can be easily cleaned with a moist cloth and a little liquid cleaning agent to remove any cleaning residues and deposits. For the interior and the clamps, it is recommended to use an appropriately sized bottle cleaner.

Note: If, after use with contra-angle or straight handpiece instruments, FG instruments are no longer firmly hold please use new silicon holders.

5. Reprocessing (machine - based)

Detailed recommendations for cleaning and care in accordance with DIN EN ISO 17664

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© NTI-Kahla GmbH Rotary Dental Instruments